


# SABO USA Face Shield KIT Assembly Instructions

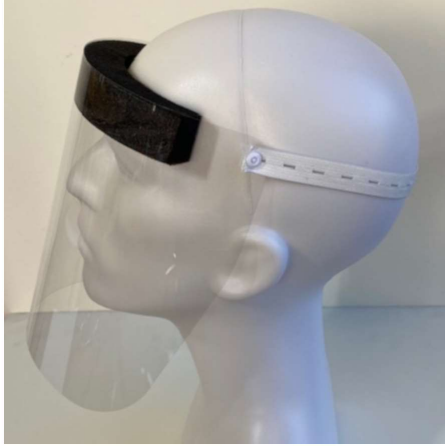

Before starting your assembly, please be sure to have a clean flat dry surface and wear white gloves provided.

1) **Foam Attachment:** Use template provided to align and register foam to lens. Begin by removing adhesive paper liner and attach foam to lens, press down firmly across length of foam for good adhesion. *Tip: Practice aligning foam to lens before removing the adhesive liner (after attaching foam to lens you will not be able to remove foam and reposition without damaging both the foam and lens).*

2) **Button Attachment:** Insert plastic male button thru front side of precut hole in lens and attach female button to back side of lens and snap together. Repeat procedure for second lens hole.

3) **Strap Attachment:** Attach elastic strap slit hole to **FRONT** side of lens buttons (opposite side of foam).

Note: For high volume face shield assemblers we recommend purchasing our "Face shield foam alignment fixture" Part # FSA-1

